# Laktator ręczny NATUREL

PL

Nr kat.053

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## Manual Breast Pump NATUREL

Natural breastfeeding meets the most important needs of the baby. It provides the baby with closeness of the mother and a sense of security. The mother's milk fully meets demand for all the elements necessary for correct development of the baby in the first months of life when the development of the little one is especially intensive. The natural milk contains also antibodies providing the infant with protection against bacterial and virus infections.

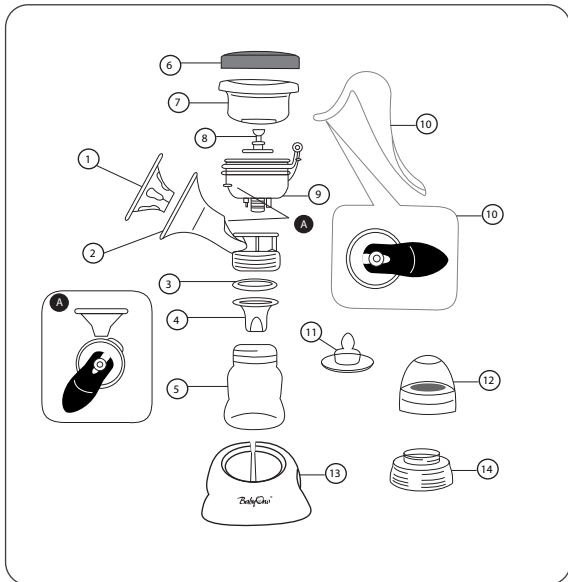
The BabyOno breast pumps enable you to return to active life by making it possible to express milk for storing and feed your baby with it when you are not present. The expressed milk, stored in a proper manner while maintaining all the hygiene principles, may be served by the infant's father or any other member of the family. Feeding by the father additionally strengthens the bond between him and the baby.

The BabyOno breast pumps are also an invaluable tool for normalising lactation. In the case of insufficient amount of milk you will stimulate your breasts to produce it more intensively. In the case of excess of milk you will express the surplus in a quick and easy way, avoiding milk stagnation and possibility of painful mastitis.

All the elements of the breast pump and bottle are manufactured from special, safe plastic intended for contact with food and not containing Bisphenol A (BPA).

## INSTRUCTIONS

**WARNING!** Read carefully the instructions for use and warnings before the first use of the BabyOno breast pump. Keep the instructions for future reference!

## BREAST PUMP COMPONENTS

1. silicone massaging pad
2. breast pump funnel
3. seal
4. silicone check valve
5. bottle
6. main body cover
7. silicone main body seal
8. suction roller
9. main body
10. handle
11. teat
12. cap with a ring
13. sealing bottle cap
14. bottle base
15. adaptor for wide neck bottles

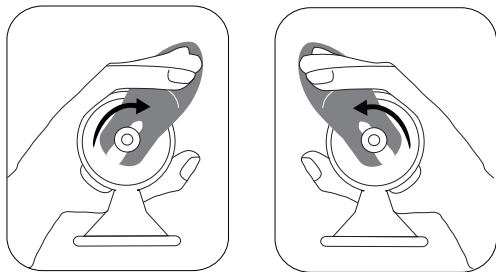
### INSTRUCTIONS FOR ASSEMBLY:

1. Place the massaging pad (1) on the breast pump funnel (2) and press it to form a whole.
2. Combine the suction roller (8) with the silicone seal (7) from the bottom.
3. Place the breast pump main body (9) on its funnel element (2) and give it a twist to the moment where the points marked in figure with A letter are aligned.
4. Next put the silicone seal (7) on the breast pump main body (9).
5. Place the handle (10) on the suction roller (8), fasten to the main body (9) of the device and put the main body cover (6).
6. Attach the seal (3) and silicone check valve (4) to the funnel and next combine the bottle (5) with the breast pump funnel (2). The breast pump is suitable for standard bottle sizes.
7. The use of the adaptor (15) allows combining the breast pump with wide neck bottles.
8. Put the assembled breast pump into the base (14).

### INSTRUCTIONS FOR USE:

1. Remember that stress has a considerable influence and impact on the amount of produced milk. Before you start expressing milk make sure to eliminate any situations stressful to you. Try to relax and always remember about hygiene: wash your hands with soap and water and your breast with water only.
2. Many women feel more thirsty during breastfeeding or expressing of milk – prepare a glass of water or anything else to drink in order not to stop expressing milk.
3. Do not be discouraged with the initial failures – you will shortly become skilled and expressing of milk will not pose a problem to you.
4. Sit down in a position comfortable to you that will enable you to lean slightly forward.
5. Take the previously disinfected breast pump (make sure that all the elements are correctly assembled). Place the breast inside the pump main body adjusting it to the silicone pad in order to place a nipple deep enough to provide its proper stimulation by petals on the pad.
6. An especially designed construction of the handle lever allows setting it at an angle suitable for the mother by positioning it to the left or right hand.

7. By pushing down the pump handle with rapid movements massage the breast and stimulate it to lactation. Next rhythmically push down the handle and release the pressure making milk flow into the bottle. Adjust the rhythm and pressure to your own needs.
8. If you do not feel any suction check whether there is any air between the breast and the silicone pad. If there is some air there, take out the breast and place it again in the pump funnel.

9. Stop expressing your milk when it ceases flowing from the breast.
10. The amount of expressed milk will vary; it depends on whether the lactation process is regulated, on a time of the day and amount of liquids that you drink. Expressing of milk from full breasts may take even 30 minutes.
11. The use of the breast pump should not be painful. If the process causes pain to you, if you can see redness and irritation of the breast, immediately consult a breastfeeding advisor.
12. Following the completion of expressing of milk wash all the pump elements and prepare them for sterilisation.

## **ASSISTANCE OF THE BREAST PUMP IN BREASTFEEDING-RELATED PROBLEMS**

- **Excess of milk** – when the breast is filled with too much milk it is difficult for the baby to correctly grasp a nipple. By expressing the excess of milk you will make the breast softer and thus you will make it easier for the baby to correctly grasp the nipple.
- **Shortage of milk** – insufficient amount of milk makes the baby more easily irritated and discourages it from sucking the breast. By using the BabyOno breast pump between proper breastfeeding you will stimulate your breast to produce more milk.
- **Nipple shape** – especially in the initial phase of breastfeeding numerous women complain about the shape of their nipples that are not sufficiently protruding which makes it difficult for the baby to correctly grasp the nipple. Regular use of the breast pump will help you to more quickly adapt the breast for feeding.
- **Sore nipples** – before you and your baby become skilled in breastfeeding the nipples may be wounded with gums of the infant who may suck them in an improper way. By using the BabyOno breast pump you give your nipples some time for healing and shortly you will be able to breastfeed the baby again.

## **CLEANING OF THE BREAST PUMP**

Disassemble all the breast pump parts (in an order opposite to the assembly instructions). Before the first use and after each subsequent use carefully clean all the pump elements in warm water with soap for children and rinse several times. The breast pump may be also washed in the dishwasher (on the top rack only), but you must remember to immediately take out and dry all its elements following the completion of the program.

After washing the breast pump and each time before its use all the part having contact with milk should be sterilised:

- silicone massaging pad (1)
- breast pump funnel (2)
- seal (3)
- silicone check valve (4)
- silicone main body seal (7)
- suction roller (8)
- main body (9)
- bottle (5), teat (11), cap with a ring (12), sealing cap (13)
- adaptor for wide neck bottles (15).

In order to do so boil them in water for 5 minutes or sterilise in a microwave or steam steriliser intended for that purpose. Never use for sterilisation any strong chemicals or any other products not intended for that purpose. Do not leave the product in disinfectants for time longer than the recommended one since it may result in its damage and/or impairment.

Store the product in a dry place at room temperature. Do not leave the product in a sun filled or hot place.

The attached teat is suitable for standard BabyOno bottles. Before each use always check by pulling into different directions whether the teat is not damaged, especially when the child already has teeth. Replace a damaged, bitten or misshaped teat with a new one! It is not allowed to make holes in the teat bigger due to the risk of damage to the product and/or child's choking. Due to hygienic reasons and to ensure safety it is recommended to replace a teat every 1-2 months.

### **For the safety and health of your child!**

#### **WARNING!**

- Always use the product under the supervision of adults. Never leave the child unattended during feeding.
- Never use bottle teats as soothers.
- The continuous and prolonged sucking of liquids may result in dental caries.
- Always check the food temperature before feeding.
- Keep all the breast pump parts away from reach of children.

Remember to assemble the sterilised parts of the breast pump and bottle with clean hands.

Product compliant with the EN 14350 standard

### **STORING EXPRESSED MILK**

**IMPORTANT: remember to properly store expressed milk! Improperly stored milk loses its nutritive value and may even do harm to the baby.**

We recommend sterile BabyOno bags for storing and freezing breast milk.

| <b>STORAGE PLACE</b>                       | <b>STORAGE TIME</b>       |                                    |
|--------------------------------------------|---------------------------|------------------------------------|
|                                            | <b>FOR HEALTHY BABIES</b> | <b>FOR PREMIES AND SICK BABIES</b> |
| At room temperature                        | 8-12 hours                | 0-1-4 hours                        |
| In the fridge ( -5°C)                      | 3-4 days                  | 24-48 hours                        |
| In the fridge freezer compartment ( -10°C) | 2 weeks                   | 1 week                             |
| In the freezer ( -18°C)                    | 6-12 months               | 3-6 months                         |

#### **Notes**

- Never refreeze breast milk. Any unused milk should be discarded.
- Do not mix defrosted breast milk with fresh breast milk.
- Only fresh breast milk should be frozen.
- It is recommended to slowly defrost breast milk. For that purpose frozen breast milk should be left in the fridge for reaching a liquid form.

- For making breast milk warm in it recommended to use hot water bath the temperature of which should not exceed 50°C.
- In order to preserve the nutritive value breast milk should not be boiled, warmed up directly on a gas burner or in a microwave!
- The temperature of the milk ready for serving should be the temperature of human body, so not higher than 37°C.

## **WE RECOMMEND ACCESSORIES FOR BREASTFEEDING MOTHERS:**

### **1. Bags for storing and freezing breast milk** (Ref.no.1039)

Manufactured from special, sterile plastic intended for contact with food and not containing BPA. The attached stickers allow marking the milk and the special construction of the bag allows its comfortable storage with no risk of spilling.

### **2. Disposable breast pads** (Ref.no. 031,031/60)

Pads with higher absorbency – designed especially for women with abundant lactation to feel comfort and freedom. Light, discreet and ultra thin.

### **3. Silicone nipple protectors** (Ref.no. 847,848)

Manufactured from super soft and naturally delicate silicone, they protect sensitive and irritated nipples during breastfeeding.

## **BABYONO ELECTRIC BREAST PUMPS**

### **Electric breast pump SENSI** (Ref.no. 051)

- High operational comfort and efficiency
- Adjustable suction strength
- Battery or mains powered

### **Sensiduo electric breast pump SENSIDUO** (No.052)

- Two-phase program naturally imitating baby's suckling action
- Adjustable, wide range of suction strength
- Exceptionally quiet operation
- Battery or mains powered

An especially designed construction of the BabyOno Naturel, Sensi, Sensiduo breast pumps enables you to easily change the manual breast pump into an electric one and the other way around, allowing you to adjust the product to your needs.

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