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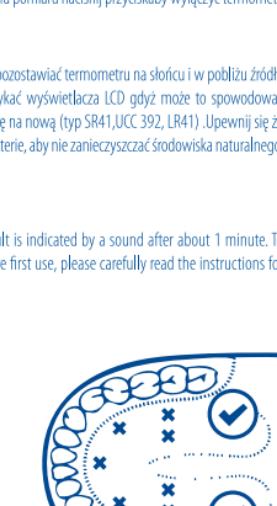
**ENG BabyOno flexible digital thermometer – Instructions for use.**

BabyOno digital thermometer allows quick, accurate and safe measurement of body temperature. The measurement result is indicated by a sound after about 1 minute. To achieve a very accurate measurement result, it is advisable to measure the temperature for approximately 5 minutes. Before first use, please carefully read the instructions for use. This thermometer allows temperature to be measured in the oral cavity, under the tongue or under the armpit.

**Specifications:**

|                          |  |
|--------------------------|--|
| Range of measurement:    | 32°C-42,9°C  |
| Accuracy of measurement: | +/-0,1°C (35,5°C-42°C)                                   |
| Display:                 | 3 digits LCD   |
| Battery:                 | 1.5 V DC (SR41, UCC 392, or LR41)                        |
| Battery life:            | 200 hours  |
| Weight:                  | approx. 10g including battery                            |
| Alarm:                   | 10 seconds of sound signals after measurement completion |
| Ambient Temperature:     | 5°C - 45°C   |
| Humidity:                | 25%RH-80%RH  |
| Storage temperature:     | -25°C - 55°C   |

**Instructions for use**

Before use wipe the end of the thermometer with 75% ethyl alcohol or immerse the end in it only, in order to disinfect the thermometer. Never immerse the whole thermometer. Press the button next to the display which will give a sound signal and show its readiness to work **BBBE**. In this time the thermometer shows the last memorised temperature result. After about 2 seconds the thermometer signals **Lo°C**. This means that the thermometer is ready to work.

1. Before measuring temperature in the mouth you should keep your mouth closed for at least 5 minutes, for more accurate measurement. The position of the thermometer's end in the oral cavity is marked on the picture „V”.

2. Measuring temperature underthe armpit: place the thermometer end under the armpit and press it tightly. When the "C" symbol stops flashing and you hear the sound signal, it means that the measurement is completed and you can read the result on the display. In order to prolong the battery life, after completion of the measurement press the turn-off button. Thermometer switches off automatically after 10 minutes.

**Precautions**

When cleaning the thermometer never immerse the whole device. The display and back cover are not water resistant. Do not leave the thermometer in the sun or near to heat sources. Do not dismantle the thermometer, except when replacing the batteries. During measurement do not touch the LCD display, because this may cause the thermometer to turn off. When the symbol **Δ** appears or it is unclear you should replace the batteries. Take off the back cover and replace the battery with a new one (type SR41, UCC 392, LR41). Make sure that the positive pole is directed upwards, and the negative pole downwards. Dead batteries should be deposited in a special container in order to prevent contamination of the natural environment. If the thermometer is not used for a long time, take the batteries out. Keep the instructions for use for informational purposes.

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